LIVING basics®

*A brand of **Shopper+**  Proudly Canadian*

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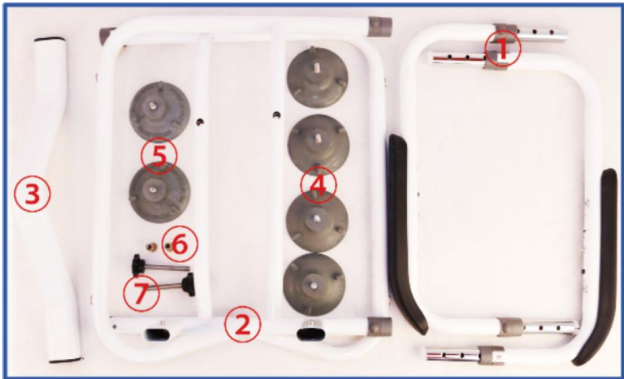
English

1. Important Safety Warnings

- Ensure all parts are intact and free of damage or cracks before use.
- Make sure the frame is stable and secure during installation and use.
- Do not apply excessive pressure to the frame; the maximum weight capacity is 330 lbs (150 kg).
- If any parts are loose or abnormal, stop using the product immediately and contact a professional for inspection.
- This product is for indoor use only and is not suitable for outdoor or damp environments.
- Always grip both handles with equal force when sitting or standing up.

2. Installation Instructions

1. Parts List: ① Upper armrest tube *2 ② Toilet main frame *2 ③ Front end left and right link pipes *1 ④ Suction cup foot pad *4 ⑤ Side clamping suction cup *2 ⑥ Side clamping suction cuplink nut *2 ⑦ Side clamping suction cup locking screw *2	
2. Remove the pin on the armrest frame and insert the upper armrest tube.	

3.Adjust to the appropriate height and insert the pin to fix it in place.

4.Insert the front link rods into the installation positions of the left and right armrests respectively.

5.Adjust to the appropriate width to ensure that the positioning ball pops out in place.

6.Rotate the long handle suction cup and install it into the mounting hole at the bottom of the armrest frame.

7. Rotate the clamp connecting nut into the short handle suction cup (note: the big end should be close to the suction cup)	
8. Rotate the fixture locking rod through the armrest frame and tighten it with the fixture suction cup installed in the previous step.	
9. Place the installed toilet stand against the toilet and gradually tighten the screws of the left and right fixtures to stabilize it.	

3. Usage Instructions

- After installation, place the toilet safety frame around the toilet, ensuring it is centered.
- When using, firmly grip both sides of the frame, making sure your movements are slow and steady.
- Avoid pushing or pulling the frame abruptly to prevent loss of balance.
- Regularly check the stability of the product and make sure the handle height suits your height.
- Follow your healthcare professional's advice when using this product.

4. Intended Use

- This product is designed to provide additional support and stability for individuals with mobility issues when using the toilet. It is especially suitable for the elderly, patients recovering from surgery, or people with walking disabilities. The frame helps users safely sit down and stand up, reducing the risk of falls.

Indications

- Elderly individuals with mild mobility impairments
- Patients in recovery with mild symptoms
- People with mild walking or balance issues
- Users who need extra support while using the toilet

Contraindications

- Users weighing over 330 lbs (150 kg)
- Patients with severe mobility impairments or significant loss of balance

5. Maintenance Instructions

- Regularly check for wear on screws, connections, and rubber feet to ensure there are no loose or damaged parts.
- If you notice cracks or corrosion on the frame or handles, stop using the product immediately and contact the supplier.
- Clean the frame with a mild detergent and a damp cloth. Avoid using strong acidic or alkaline cleaners.
- Regularly check the stability of the frame, especially after frequent use.

6. Storage and Placement Instructions

- When not in use, store the toilet safety frame in a dry and well-ventilated place.
- Avoid storing the product in damp or direct sunlight environments to prevent material degradation.
- If not used for a long time, disassemble the frame and store it in its original packaging for proper safekeeping.

7. Product Specifications

Product Name: Toilet Safety Frame

Model: LB-YB-2014

Standard: Canadian Standard (SOR/98-282)

Maximum Weight Capacity: 150 kg / 330 lbs

Adjustable armrest height: 65-77CM

8. Warranty Terms

Warranty Period: 90-day warranty service. If any quality issues or defects are found during the warranty period, please contact customer service promptly.

9. Contact Information

Brand Holder: Shopper+ INC

Brand Holder Address: 2110 52E AV, LACHINE, QUEBEC, H8T 2Y3, MONTREAL, CANADA

Email: support@shopperplus.com

Phone: 514-631-5216

Manufacturer: Foshan Yoob Medical Instrument Co.,Ltd

Manufacturer Address: Factory 804, Building 4, Phase 9, Tianfulai Industrial Zone, No. 12, Rongli Xinfu Road, Ronggui Street, Shunde District, Foshan City, Guangdong ,China

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Fabricant : Foshan Yoob Medical Instrument Co.,Ltd