LIVING basics®

*A brand of **Shopper+** 🍁 Proudly Canadian*

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English

1. Important Safety Information

Before using this product, carefully read and understand the following safety warnings:

- **Warning:** Maximum weight capacity is 300LBS/136Kg. Do not exceed this limit.
- **Warning:** Before use, ensure the chair's height and seat angle are adjusted to fit your needs and that all adjustment mechanisms are securely locked.
- **Warning:** Avoid using this chair on uneven surfaces or in damp environments to prevent slipping.
- **Warning:** Regularly check the rubber foot pads and all supporting components for wear. Replace them promptly if any wear or failure is detected.
- **Warning:** Before each use, ensure all fixed components are intact and securely tightened.
- **Warning:** Store the chair in a dry place and avoid exposure to extreme temperatures and humidity.
- **Warning:** Always maintain a firm grip on the armrests when using the chair.
- **Warning:** Use this chair under the guidance of a qualified physician or therapist.
- **Warning:** Do not apply lateral force to the chair. Avoid applying any pressure to the sides of the chair to prevent unnecessary damage or risk of falling.

2. Installation Instructions

1. Parts List:

- ① Release Button
- ② Support frame
- ③ Base Plate
- ④ Bucket
- ⑤ Front wheel (optional)
- ⑥ Height adjustable
- ⑦ Tube Foot Pad
- ⑧ Hook

2. Folding and Unfolding:

- Folding: Release the locking mechanism or hinge, then gently fold along the joint.
- Unfolding: Follow the reverse process of folding, ensuring all locking mechanisms and hinges are securely engaged.

Note: Pressing the button will fold the product, making it suitable for compact storage without taking up too much space.

3. Height Adjustment:

Press the adjustment button or release the locking mechanism to adjust the chair to the desired height. After adjustment, ensure all parts are securely locked in place.

Note: When adjusting height, ensure that each leg's adjustment button is fully engaged in the corresponding hole, making sure all legs are locked in place and at the same height.

4. Assemble the seat plate:

Align the four C-clamps at the bottom of the seat plate with the side tubes and press down so that the seat plate snaps onto the side tubes.

5. Bucket Replacement:

The bucket is removable and replaceable. Align the new bucket with the slot, ensuring it is fully secured.

3. Usage Instructions

Preparation Before Use: Ensure the rubber foot pads are securely in contact with the ground. Hold the armrests firmly and maintain balance during use.

Usage Scenarios:

- **Toilet Assistance:** Used in conjunction with a toilet, serving as a commode chair to help users sit down and stand up.
- **Portable Toilet:** Functions as a commode chair, with the option to detach or replace the toilet bowl as needed.
- **Standard Walker:** Used as a conventional walking aid to help users walk indoors and outdoors.
- **Rolling Walker:** After installing wheels, the chair can be used as a rolling walker to assist users with movement.
- **Toilet Seat Elevation:** Adjusts the seat height to make it easier to stand up from the toilet.
- **Shower Chair:** Equipped with a waterproof seat, suitable for use in the bathroom.

Note: Regularly check screws to ensure they are fully tightened to maintain the product's stability and safety.

Prohibitions: Avoid applying any force or pressure to the sides of the chair to prevent damage or unsafe conditions.

4. Intended Use

This chair is designed to provide multifunctional support and comfort for individuals with limited mobility, suitable for the following situations:

- **Elderly Assistance:** A lightweight and stable device to help elderly or disabled individuals stand and walk, with strong weight-bearing capacity and non-slip armrests.
- **Shower Commode Chair:** Waterproof design suitable for use in bathrooms, functioning as both a shower chair and a commode.
- **Post-Surgical Rehabilitation:** Provides support to help users gradually regain mobility.
- **Daily Use:** Offers support and comfort in and around the bathroom, reducing the risk of falls.

Indications:

- Individuals with limited mobility: such as the elderly, those recovering from surgery, or individuals with mild gait instability.
- Patients in rehabilitation: needing additional support to assist with standing and walking.

Contraindications:

- Severe overweight individuals: Users exceeding the maximum weight capacity of 300LBS/136Kg should avoid use.
- Severe lower limb injuries: Should use other medical equipment under medical supervision.

Precautions:

- Ensure the chair is properly adjusted and securely locked before use.
- Regularly maintain all components of the chair to ensure proper function.
- Exercise caution when using on wet or uneven surfaces to prevent slipping.
- Follow the recommendations of a physician or therapist when using this chair.

5.Maintenance and Care

- **Routine Checks:** Regularly inspect the chair for wear, especially the rubber foot pads and supporting components.
- **Cleaning:** Wipe the chair with a damp cloth. Do not use strong chemicals or submerge the chair in water.
- **Storage:** Store the chair in a dry place when not in use, avoiding exposure to extreme temperatures and humidity.

6.Compliance

This chair complies with Canadian regulation SOR/98-282, which outlines the requirements for medical and assistive devices.

7.Product Specifications

- **Product Name:** Multifunction Aluminum Foldable Commode Chair
- **Model:** LB-BME-AC816
- **Main Materials:** Aluminum alloy, plastic seat, rubber foot pads
- **Maximum Weight Capacity:** 300LBS/136Kg
- **Seat Height Adjustment:** 45-62cm
- **Seat Dimensions:** Width 44cm x Depth 40cm
- **Maximum Armrest Height:** Adjustable height 78-95cm

8.Warranty Terms

- **Warranty Period:** The product comes with a 1-year warranty. If any quality issues or defects are found within the warranty period, please contact customer service.
- **After-Sales Service:** For warranty service or any inquiries, please contact our customer service team.

9.Contact Information

- **Brand Holder:** Shopper+ INC
- **Brand Holder Address:** 2110 52E AV, LACHINE, QUEBEC, H8T 2Y3, MONTREAL, CANADA
- **Email:** support@shopperplus.com
- **Phone:** 514-631-5216
- **Manufacturer:** Sunshine Healthcare Co., Ltd
- **Manufacturer Address:** No. 31 Jihua Eastern Road, Nanhai, Foshan, China, 528000

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- **Fabricant** : Sunshine Healthcare Co., Ltd