

LIVING basics

*A brand of **Shopper+**  Proudly Canadian*

Folding 4-Wheel Rollator Walker Instruction Mannual

SKU:LB-FSD-FST935A



Safety Notice:For your safety, please read this user manual carefully and completely before use.

Important Precautions: Before using your Rollator, you must read and fully understand all of the following safety instructions. This product is a personal mobility aid. Any improper use may cause serious injury to the user or others.

ABOUT PRODUCT




- 1: Hand Grip
- 2: Brake Handle
- 3: Backrest
- 4: Main Frame
- 5: Flip-up Seat
- 6: Cup Holder
- 7: Storage Bag
- 8: Folding Strap
- 9: Cane Holder
- 10: Rear Wheel
- 11: Front Wheel

Product Assembly Guide



1: Open the packaging and carefully remove all components.

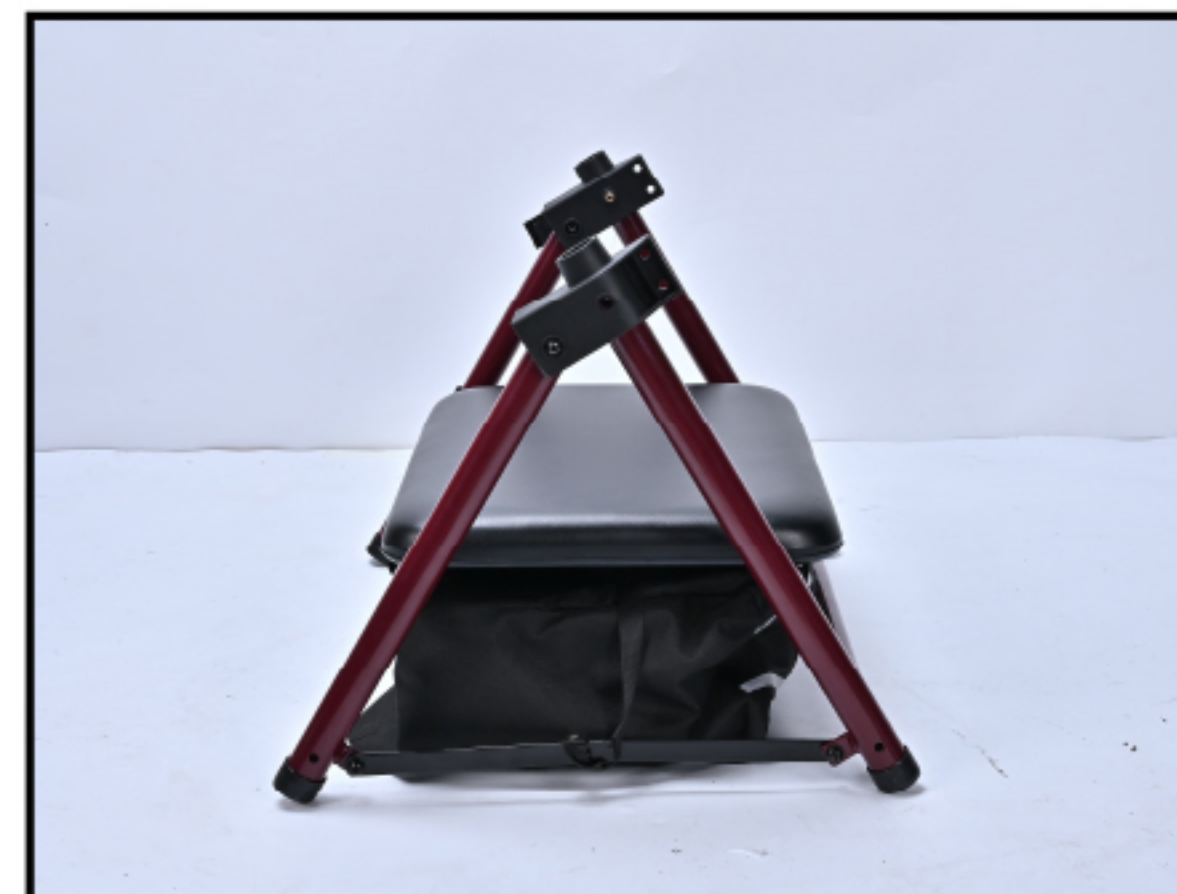

2: Unfold the main frame: Expand the main frame and flatten the seat as shown in the illustration above. Ensure the frame is fully extended to its maximum angle.



3: Please use the screws indicated above for assembly.



4: Insert the front wheel (labeled position 1) into the main frame (labeled position 1). Align the holes, then insert screw #2 into the hexagonal opening (ensure the screw aligns with the hexagonal hole). Add the washer and the knob nut (the concave side of the washer should face the main frame), then tighten securely.



5: Insert the front wheel (labeled position 2) into the main frame (labeled position 2). Align the holes, then insert screw #2 into the hexagonal opening (ensure the screw aligns with the hexagonal hole). Add the washer and the knob nut (the concave side of the washer should face the main frame), then tighten securely.



6: Insert the rear wheel (labeled position 3) into the main frame (labeled position 3). Align the holes, then insert screw #2 into the hexagonal opening (ensure the screw aligns with the hexagonal hole). Add the washer and the knob nut (the concave side of the washer should face the main frame), then tighten securely.



7: Insert the rear wheel (labeled position 4) into the main frame (labeled position 4). Align the holes, then insert screw #2 into the hexagonal opening (ensure the screw aligns with the hexagonal hole). Add the washer and the knob nut (the concave side of the washer should face the main frame), then tighten securely.




8: Insert each handle into the main frame (pay attention to the orientation of the handgrips). Align the holes, then insert screw #1 and tighten securely.



9: Secure Brake Cables: Snap both brake cables into their respective clips.

Important Note: Ensure all caster wheels are adjusted to the same height level.

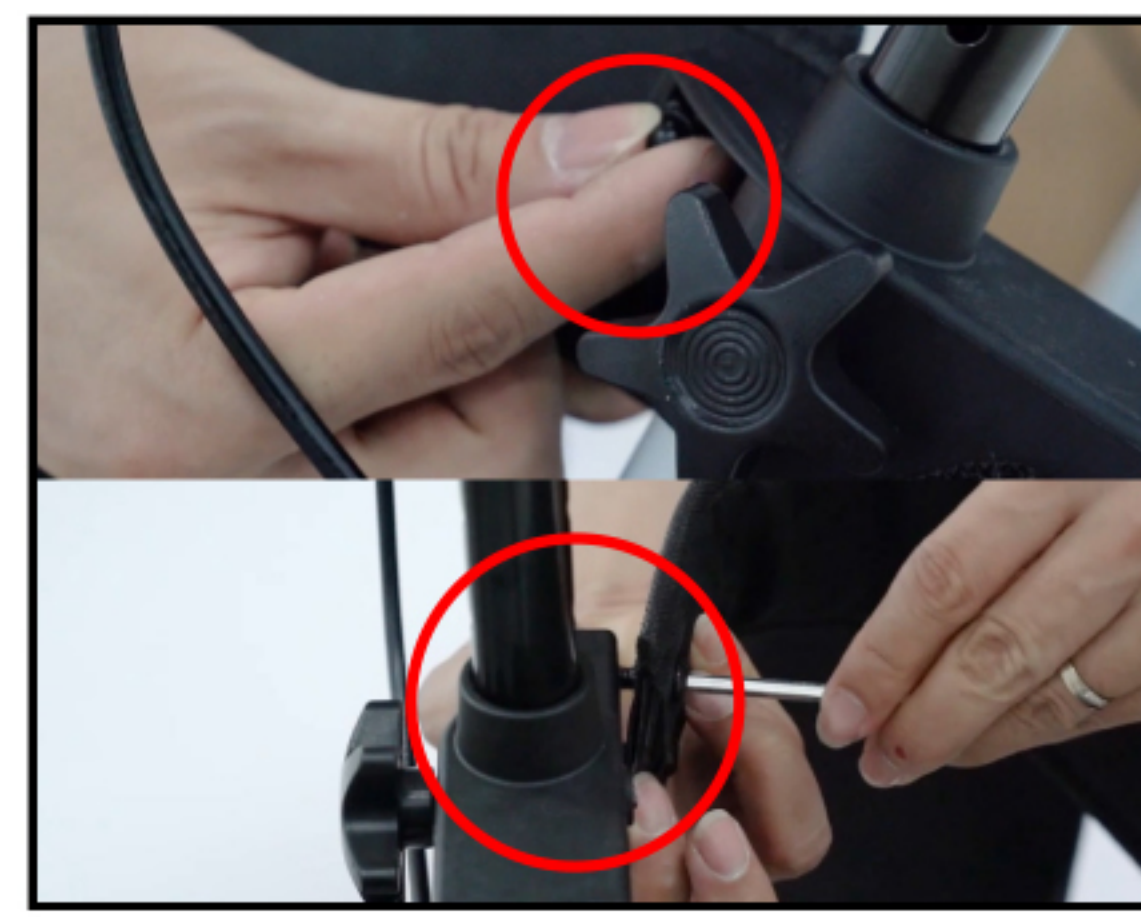

10: Install Backrest: Insert two (2) #3 screws through the upper and lower holes of the backrest, as shown in the illustration.

11: Install Backres: On the opposite side, place the nuts, then use the L-shaped wrench to tighten the screws, as shown in the illustration. (Repeat steps 10 and 11 to install the other side.) Attach Cane Holder: Install the cane holder tray onto the rear wheel leg tube at labeled position 4. Ensure it snaps securely into place and will not detach. Attach Cup Holder Pouch: Install the cup holder pouch onto the other side of the frame. Adjust its position, then firmly attach the Velcro straps, ensuring the cup holder pouch is secure.

Folding the Rollator



1. Lift the Seat: Flip the seat upwards.



2. Pull the Folding Strap: Pull the folding assistance strap upwards.



3. Complete Fold: The rollator is now folded.

Safety Precautions

1. Before each use, ensure the rollator is fully opened, the seat is down, and the folding bar under the seat is locked in the down position.
2. Check that both height adjustment knobs are fully tightened and set to the same height. Test the rollator's stability before applying your full weight to the hand grips.
- 3: Always test the brakes (loop locks) before use. When the brakes are engaged, the lock bar on the rear wheels should prevent them from turning. If the wheels turn while the brakes are locked, do not use the rollator. Contact your dealer for adjustment. When the brakes are released, the wheels should turn freely.
- 4: Always engage the brakes before sitting. Do not use the seat on an incline or uneven ground.
- 5: Replace tires if they are significantly worn or damaged.

This is a walking aid only. It is NOT to be used as a transportation device.
DO NOT allow anyone to push you while you are seated on the rollator.

Brake & Lock Operation

Important: Always apply and release the brakes on both sides at the same time.

1. To Slow Down or Make a Temporary Stop (Service Brake)




Gently pull both brake handles upwards towards you. The more you pull, the slower the rollator will move.

2. To Lock the Brakes (Parking Brake)

Press both brake handles down firmly with your palms until you hear and feel them "click" into the locked position. Always check that both brakes are locked before attempting to sit.

3. To Release the Locked Brakes

Pull both brake handles upwards until they unlock and return to the neutral, original position.

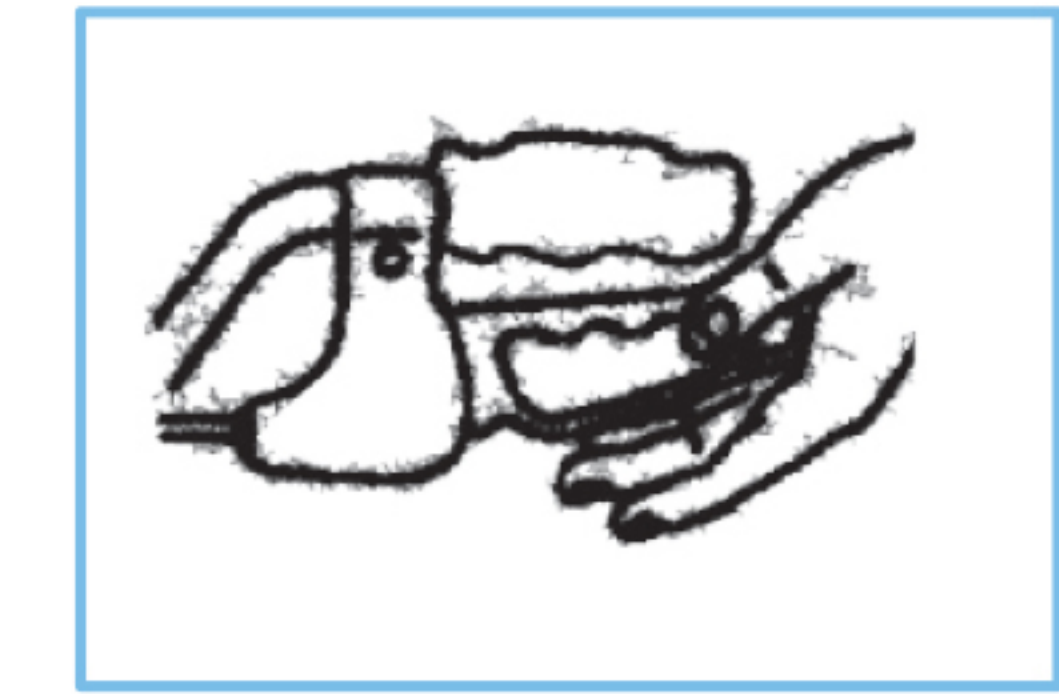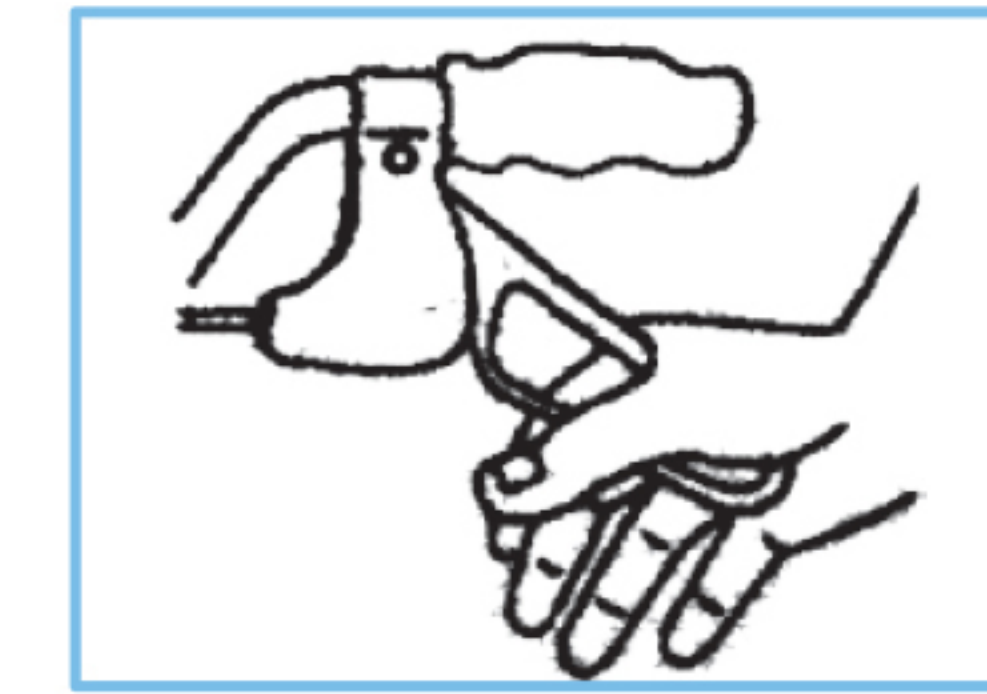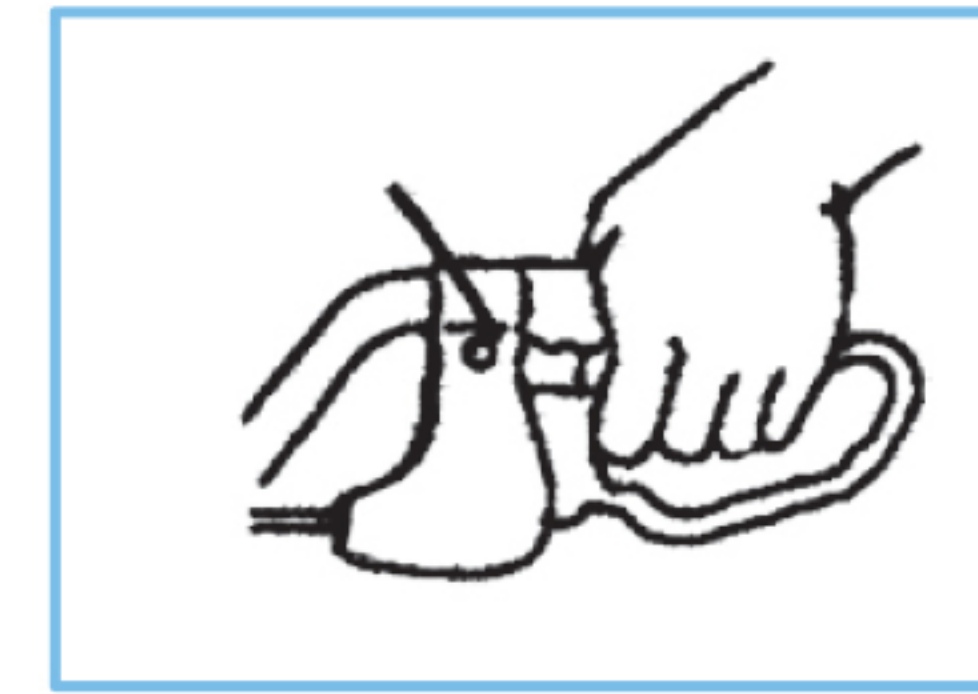

Instructions to adjust the Hand Brake System

Important Safety Information

The brake system installed on this rollator meets all required quality and safety standards. For optimal performance, the brakes have been preset at the factory. However, adjustments may be necessary to suit your personal preference or to compensate for normal wear over time. Regularly check the brake performance and adjust as needed following the instructions below.

Hold firmly in place do not turn.

To tighten the hand brake adjustment.
"turn the nut clockwise"

To loosen the hand brake adjustment.
"turn the nut counter clockwise"



General Safety Instructions

Weight Capacity: DO NOT exceed the maximum user weight capacity or the maximum load capacity for the storage bag as specified.

Not a Transport Device: This product is NOT a wheelchair. NEVER allow anyone to push you while you are seated in the rollator. DO NOT use it to transport people or heavy objects.

Proper Posture: Always walk within the frame of the rollator, not behind it. Keep your body centered between the handgrips to ensure stability.

Both Hands on Grips: Keep both hands on the handgrips at all times when walking to maintain full control and access to the brakes.

Do Not Overload: DO NOT hang heavy items, purses, or bags from the handgrips or the side of the frame. This can cause the rollator to become unstable and tip over.

Be Aware of Surroundings: Watch out for potential hazards on the ground, such as rug edges, electrical cords, thresholds, and spills.

Brake & Seating Safety

ALWAYS LOCK BRAKES BEFORE SITTING: Before you attempt to sit on the seat, you must engage the parking brakes by pushing both brake levers down firmly until they “click” into the locked position. Test that the rollator will not move.

Sit Safely: Turn around and back up to the seat. Use the handgrips for support and slowly lower yourself onto the center of the seat.

Do Not Self-Propel: NEVER attempt to move the rollator by "scooting" with your feet while you are seated.

Maintain Balance While Seated: DO NOT lean forward or to the side to reach for objects while seated, as this can cause a loss of balance and lead to a fall.

Terrain & Environmental Limitations

NEVER Use on Stairs or Escalators: This rollator is strictly forbidden for use on stairs, escalators, or moving walkways.

Use Caution on Slopes: Exercise extreme caution on ramps, inclines, and declines. Use the brakes to control your speed when going downhill.

Avoid Hazardous Surfaces: Be careful on wet, slippery, or icy surfaces. Avoid using the rollator on uneven terrain, loose gravel, thick carpets, or soft ground where wheels can get stuck.

Maintenance & Care

Regular Cleaning: Clean your rollator periodically with a damp cloth and a mild, non-abrasive cleaner. Dry thoroughly.

Check Fasteners: On a monthly basis, check all screws, nuts, and bolts to ensure they remain securely tightened.

Professional Repair: If any part or component—especially brakes and wheels—becomes damaged, worn, or loose, stop using the rollator immediately. Contact your dealer or a qualified professional for repair. Only use manufacturer-approved original replacement parts.

Product Name : Foldable Rollator

Model : LB-FSD-FST935A

Manufacture Date : 202603

Maximum Weight Capacity : 300 lbs/136 kg

Brand Owner : Shopper+ INC

Brand Owner Address : 2110 52E AV, LACHINE, QUEBEC, H8T 2Y3, MONTREAL, CANADA

Email: support@shopperplus.com

Phone : 514-631-5216

Manufacturer :

Manufacturer Address :