# LIVING basics

## Foldable Lightweight Wheelchair User Manual

LB-BME-4615

## Warning

1. Practice to use the wheelchair with companion before use.
2. Do not lean backwards in the wheelchair without sway bars.
3. Do not tilt the wheelchair to the right side or left side or lift the wheelchair while using it.
4. Do not brake the wheelchair while the wheel is rolling. If without companion, the user can grip the hand-rim tightly to stop.
5. Do not stand on the plate, which will turn the wheelchair over.
6. Do not modify the wheelchair without our technician's permission. The warranty is not valid in the case of unauthorised modification.
7. Do not use the wheelchair on an escalator.


## Main Components of Transit Chair

# Product Usage

Mobility aid for differently disabled, the sick, the old and the weak., the sick, the old and the weak.

## Quickly unfold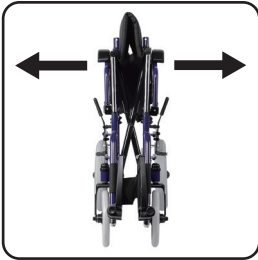

1. Hold the handgrips with two hands and pull them apart.

2. Press down the cushion

3. Lift the backrest up

4. Lock the backrest

5. Unfold the pedals and lock

6. Install the nylon strap

## Warning!

Do not pull the handgrips too hard while unfolding in case of damaging the parts; do not hold the seat tube with fingers while pushing down the cushion in case of pinching your fingers. (Picture 2)

(Picture 2)

## Folding the Wheelchair

1. Untie nylon belt

2. Unlock the buckle, then fold the pedals to the sides


3. Unlock the backrest

4. Fold down the backrest

5. Lift the cushion up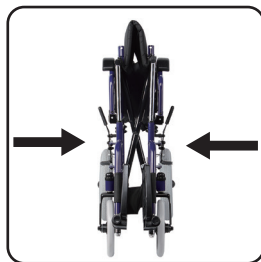


6. Finish folding

## Install the foot plate

1. Put your footplate on the raised metal pieces by aligning it with the holes

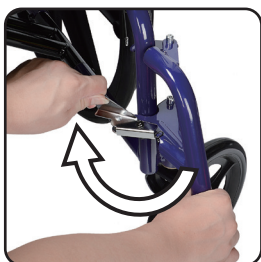2. Rotate the foot pedal inward and lock

3. Finish

## Install the Anti-tip wheel

Insert the anti-tip wheel into the tail-pipe of the wheelchair.  
Then use the pin to secure the anti-tip wheel.

## Flip up Armrest

Press the plastic lock device and pull up the armrest, then it will flip up

# How to sit down and standing up from a wheelchair?

## Sitting down into a wheelchair

- 1.Lock the brakes on both sides.
- 2.Flip up the foot plates on both sides.
- 3.Hold the arm rests with both hands and slowly sit down into the seat.
- 4.Place feet firmly on the footrests

## Standing up from a wheelchair

- 1.Lock the brakes on both sides.
- 2.Take your feet off the foot plates.Take your feet on the floor.
- 3.Hold the arm rests with both hands and slowly stand up from the seat.

## Note:

Lock the brakes on both sides while getting on or out of

## warning:

Do not step on footplate while getting on or out of .

## Safe Usage of Crossing Obstacles

The companion must hold the handgrip tightly and press the pedal of bottom tube to lift the front castors, then cross the obstacle.

## Wheelchair Transportation

We recommend that you do not sit on your Wheelchair when in a vehicle, but transfer to a vehicle seat. A Wheelchair secured in a vehicle will not provide the equivalent safety level and security as the vehicle's seating systems. However, when transfer is not practical or impossible, ensure the following:

- 1) Check that the vehicle is suitably designed, insured and equipped to transport a passenger seated in a Wheelchair
- 2) Remove any part of the Wheelchair (accessories, etc.) that can be easily detached and store it in the vehicle's luggage hold
- 3) Ensure the Wheelchair is transported in a forward facing direction. Rearward facing is only acceptable if the passenger's head and back can be supported by a suitable bulkhead

## Maintenance

1. Check the bolts of every parts of the wheelchair to see if they are loosen after being used for a period. Tighten the loosen ones to avoid the hidden danger.
2. Add lubricating oil to the moving parts each week in case of inflexibility using.
3. Use soft dry cloth to clean the moisture and the contamination on the wheelchair to avoid the rusting.
4. Keep the cushion and back pad dry and lean to avoid the skin infection.

# Attention

- 1 . The transit chair is used indoor or outdoor flat ground. It is not suitable to use it for a long time on a sloping surface.
2. The brakes are used to stabilize the transit chair while people are sitting in the transit chair, getting off the wheelchair or stopping the wheelchair on an uneven road. Do not brake the wheelchair while the wheelchair is moving. The transit chair might turn over if braking the transit chair when it is moving on a downhill path.
3. Please don't stand or apply force on the footrest while getting on or out of the chair.
4. If the chair doesn't have an anti-roll device installed, please don't incline the back rest in case the fall happens.
5. If you change the direction of the chair in downhill path, it will cause instability and possibly even a fall.

# After-Sale Service

1. If there is any quality problem not caused by intentional actions in one week since the day the product be sold, the company is responsible for return, replacement, repair;
2. Free services will be provided for any quality problem under regular services within the maintenance period

# The following is not covered by warranty:

1. Broken front castors and tyres; broken or split upholstery and armrest.
2. Breakages by improper use.
3. Breakages by unforeseeable natural disasters.
4. Regarding quality problems beyond the warranty period, users should report such to the company who supplied the chair.
5. Our company provide repair of parts with reasonable cost according the invoice and guarantee card. If user can't provide invoice, the maintenance period should be identify as one extra month after manufacture date

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